



ALICAM® Summary Administration and Retrieval Instructions for Use

Failure to properly prepare the patient may severely compromise image quality and impair the diagnostic capabilities of the exam.

PATIENT PREPARATION

- The dog should not eat any food or treats or play with chew toys for 16 to 24 hours before and 8 hrs after administration
- Provide access to ample water for duration of examination

CAPSULE ACTIVATION

- Open package and remove capsule from magnet
- Capsule will blink twice and then stop blinking
- Wait 30 seconds, and LEDs should activate and blink rapidly
- Administer ALICAM once capsule is blinking rapidly
- If LEDs do not flash 20 seconds or more after removing the capsule from the package, place the battery-end of the capsule against the magnet in the package for 30 seconds, and remove again
- Place serial number sticker from packaging onto the included plastic vial

CAPSULE ADMINISTRATION

- Do not coat the capsule with food and do not allow dog to chew capsule.
- Administer ALICAM® within 10 minutes of activation
- Place capsule on the back of the tongue, then hold the mouth shut and tilt the head back until the dog swallows
- If the patient has a history of vomiting, it should be observed until the capsule has passed out of the stomach
- If the capsule is vomited up soon after swallowing, it can be rinsed with water and re-administered
- An anti-emetic should be considered to decrease the likelihood of vomiting if this may be an issue

DISCHARGE INSTRUCTIONS

- Review instructions for recovering and cleaning the ALICAM® capsule with your client.
- Instruct your client to either return ALICAM® in the vial to your clinic or to return it directly to Infiniti Medical
- Send the owner home with: ALICAM® Owner Instructions, the Retrieval vial with serial number sticker, and the prepaid return envelope (if client is to mail the capsule back to Infiniti Medical directly)
- Instruct owner to contact your office if capsule is not retrieved after 72 hours
- The majority of capsules will be retrieved within 72 hours however in some situations it may take 1-2 weeks to recover the capsule

CAPSULE RETURN

- Complete the patient information form upon capsule administration at www.alicamvet.com/caseform
- The capsule should be returned in the vial provided via prepaid return FedEx Envelope to Infiniti Medical
- Contact Infiniti Medical if the envelope is misplaced
- ALICAM® will be processed within 1-2 business days following receipt of BOTH the patient information_form and clean capsule
- A detailed report link will be directed to email address on patient information form.

This product is intended for veterinary use only. It is not for human use.

Refer to Full Instructions for Use for detailed information.

Contact technical support at 650-327-5000 or alicamsupport@infinitimedical.com with administration or return questions.

CE

This product is intended for veterinary use only. It is not for human use.
Refer to Full Instructions for Use for detailed information.
Contact technical support at 650-327-5000 or alicamsupport@infinite-medical.com with administration or return questions.